

As safe as a child's car seat, as comfortable as a cot, as accurate as a seca.



The seca 727 electronic baby scale always thinks of everything.


seca 727

Capacity	20 kg
Graduation adjust.	2 g/5 g < 10 kg > 5 g
Size (WxHxD)	551 x 145 x 320 mm
Weight	6,5 kg
Operation mode	Mains/rechargeable batteries
Colour: white	


Every gram counts when weighing babies and small children. The seca 727 digital baby scale does just that. It works on the same principle as highly sensitive incubator scales and is also built just as compactly: Baby basin and lower compartment form a stable, anti-rocking unit.

Ergonomic perfection. The consummate design of the seca 727 follows its function. The weighing basin is made of dermatological plastic that feels warm even to a baby's tender skin, is effortless to clean and disinfect. The basin's shape is ergonomic perfection so that the baby lies quietly, safely and comfortably. If you want extra underpadding, it's easy to just tare its weight out.

Absolute stillness. Babies kick and squirm. That's why the seca 727 has built-in shock absorption that compensates for movements, providing an accurate weight every time. The curve illustrated below

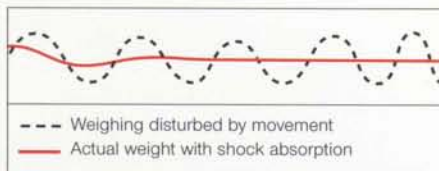

shows how this shock absorption works. The averaged weight is shown on the large, well-hit LED display. The weight is indicated

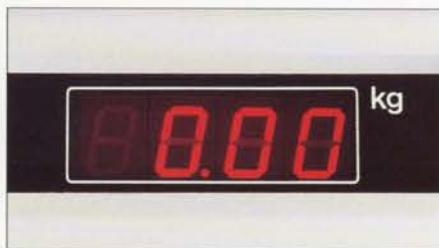


in such minute graduations that even a drop of water would be noticed: Up to 10 kg, the graduation can be adjusted to either 2 g or 5g. Above 10kg, the graduation is always 5g.

Designed with every detail in mind.

By pressing the HOLD key, the weight can be held in the display for two minutes, allowing you to devote your time to the baby first. The seca 727 also has a PC interface option for transferring weight readings directly into the patient's medical records.

Functionality and accuracy.

With seca, accuracy counts above all: The diagram below illustrates just how accurate the seca 727 is.

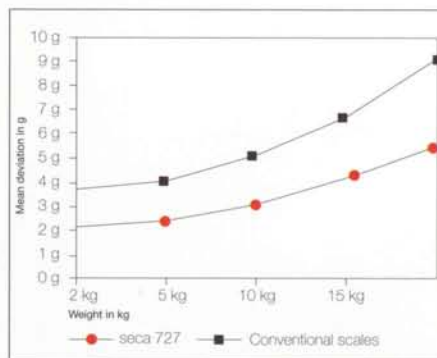

And don't forget that baby scales are also a tool used by doctors and nursing staff where quick and simple handling is essential. The seca 727 is an example of well-conceived functionality: Three large keys control all functions. When the scale is not in use, it turns itself off automatically: If the scale is running on mains power supply, it turns off after 10 minutes, if it is battery operated, after 60 seconds. No wonder that no other brand is seen in hospitals, paediatric departments and doctors' practices more than seca.

seca
Precision for health